


SaniCap™


Disinfects Tagging Gun Needles

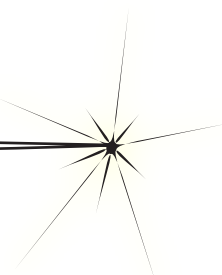

Direction for Use

Instrucciones para el Uso

1
REMOVE
SaniCap™ from
cardboard tube

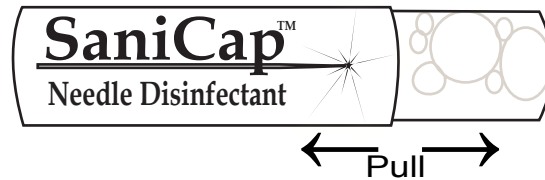


1
REMUEVA
SaniCap™ del paquete

2
POP
inner casing by
squeezing middle
firmly.




2
APRETAR
entre los dedos hasta
hacer estallar la envoltura
de adentro.

3
INSERT
needle into swab end
of SaniCap™.

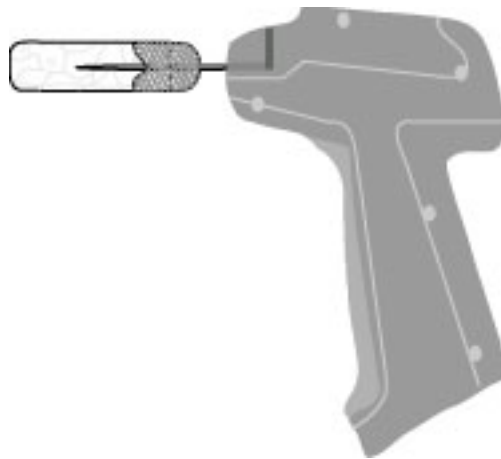

3
INSERTE
la aguja a través de
SaniCap™.

4
DISCARD
SaniCap™ in trash

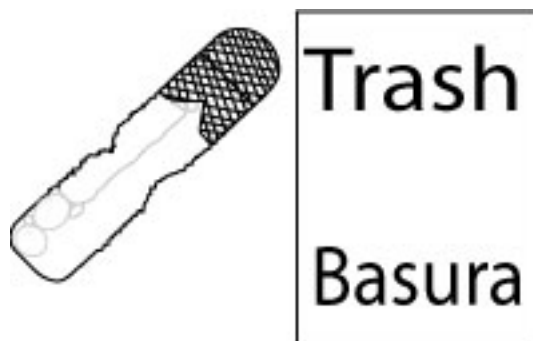

4
PONGA
SaniCap™ en la basura.